**Tkmid<sup>®</sup>**

# Single-use Medical Face Mask For Children

EN14683:2019+AC:2019 Type I

3-lagig mit Ohrenband  
3 Layers With earloop  
3 couches avec contour d'oreille  
3 strati con clip per l'orecchio  
3 vrstvy se sluchátkem  
3 lag med øreprop

**BFE ≥95%**  
**50 PCS**  
**14.5cmx9.5cm**

-  Medizinische Einweg-Gesichtsmaske
-  Single-use Medical Face Mask
-  Masque facial médical à usage unique
-  Maschera medica monouso
-  Jednorázová lékařská maska na obličej
-  Engangs medicinsk ansigtsmaske

MedPath GmbH

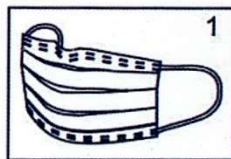
Mies - van - der - Rohe - Strasse 8 , 80807 Munich , Germany

Anhui Tiankang Medical Technology Co., Ltd

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The product should be stored in a non-corrosive gas, dry, well-ventilated and clean environment. Avoid direct sunlight. Keep away from fire and combustible materials.

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### **Intended use**

The product intended to limit the transmission of infective agents between staff and patients during surgical procedures and other medical settings with similar requirements.

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#### **Caution**

- The product is disposable, non-sterile and valid for three years. Before use, please pay attention to the expiration date of the product. Do not use after expiration.
- For single use only. No reuse.
- When wearing a disposable mask correctly, you should replace the mask when you smell odor or feel short of breath; if the mask is damaged, humid or dirty, you should replace it immediately.
- Used masks should be disposed of in accordance with the requirements of hospitals and environmental protection departments.
- Do not use mask that is split, disintegrated or with foreign objects.

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#### **Contraindications**

The product contains non-woven fabrics. Use with caution to those who are allergic to non-woven fabrics.

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GreenSun Germany GmbH